



MATERIALS PROVIDED

Pre-filled 10% formalin filled specimen container and specimen transport bag.

CLINICAL PROCEDURE

1. Confirm patient identity.
2. Label all formalin containers with patient name, date of birth, and source site.
3. Place formalin container(s) into the specimen bag. Multiple containers from the same patient can be submitted in the same bag.
4. Place paper copies of face-sheets and requisitions into pouch on specimen bag.
5. Place completed biohazard bag into provided laboratory box/pack for shipping/pickup.
6. Seal the mailer and place a shipping label on the outside.
7. Place laboratory box/pack in the designated area for shipping/pickup.
8. If shipping/pickup is needed and not already scheduled, please call the lab.