



MATERIALS PROVIDED

12 tissue cassettes labeled with prostate sites, two pre-filled 10% formalin containers, biohazard specimen bag, shipping container, and absorbent spill pad.

CLINICAL PROCEDURE

1. Confirm patient identity.
2. Label both formalin containers with patient name and date of birth.
3. Place the biopsy specimens into the appropriate pre-labeled cassette.
4. Place the left biopsy tissue cassettes into the specimen container labeled "Left." Place the right biopsy tissue cassettes into the specimen container labeled "Right."
5. Place both formalin containers into the specimen bag with absorbent pad.
6. Place paper copies of face-sheets and requisitions into pouch on specimen bag.
7. Place completed biohazard bag into provided laboratory box/pack for shipping/pickup.
8. Seal the mailer and place a shipping label on the outside.
9. Place laboratory box/pack in the designated area for shipping/pickup.
10. If shipping/pickup is needed and not already scheduled, please call the lab.