



MATERIALS PROVIDED

1 Urine Cup | 1 Urine Monovette Tube | 1 Biohazard Bag | Lab Pak Mailer

CLINICAL PROCEDURE

1. Don appropriate PPE.
2. Remove the safety label from the top of the cup to expose the port area.
3. Remove the cap from the urine Monovette syringe.
4. Insert the urine Monovette syringe into the transfer syringe port.
5. With slight pressure, push down on the needle free port area where it will secure the needle free syringe into the needless port. (If urine volume does not meet the port area, attach urine straw to the end of the syringe prior to accessing the port)
6. Pull up on the plunger and fill syringe to the fill line.
7. Replace the cap onto the needle free syringe.
8. Break off the plunger.
9. Label syringe with patient first and last name, DOB, date of collection.
10. Place into biohazard bag.
11. Remove gloves and perform hand hygiene.
12. Place all samples collected for the day into the provided Lab Pak mailer or large, red Biohazard bag. Seal the mailer and place a shipping label on the outside, or prepare for courier.
13. Do not hold samples. Ship all samples collected on the same day as collection.